

How to use the side stream CO2

1. Please check all the side stream CO2 accessories prepared. There are 4 items:
 - 1) Water Trap – Fig. 1
 - 2) Sampling Cannula for nasal use – Fig.2
 - 3) Sampling Cannula for Ventilator use – Fig.3
 - 4) T-type tube –Fig. 4

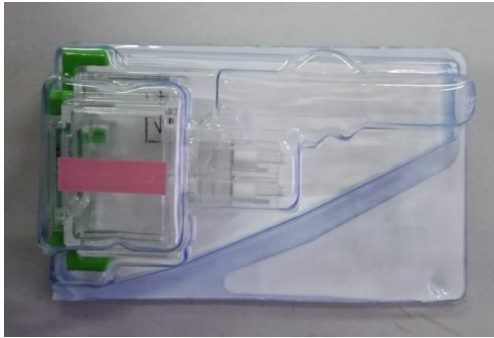


Fig. 1



Fig.2



Fig. 3



Fig.4

2. Install water trap into the trap receptacle.



The sampling line
connector

3. If it is routine monitoring, you only need to connect “*Sampling Cannula for nasal use*” between connector on the water trap and patient’s nose.



4. If it needs to be used with ventilator, please use T-type tube and “*Sampling Cannula for Ventilator use*” to connect the water trap and ventilator.



5. Once connect all the cables, the monitor will begin to monitoring CO₂.

Notice: Some monitors may need to do zero calibrating, please refer to the operation manual.

(END)